

MEDAPRINT FOAM[®]

Foaming disinfectant for dental impressions

MEDAPRINT FOAM is a disinfecting spray especially developed for the quick and easy cleaning and disinfection of dental impressions. Its foaming action allows a precise and easier application on the dental prints.

MEDAPRINT FOAM has been formulated to ensure a broad antimicrobial activity spectrum without affecting the dimensional accuracy of the impression materials.

It is fully compatible with alginate, silicone, polyether and hydrocolloids.

Properties

- Foam spray for easy and precise application
- Broad antimicrobial activity spectrum in 15 minutes
- Compatible with impressions from alginate, silicone, polyether and hydrocolloids
- Effectively removes organic residues
- Bactericidal, fungicidal, tuberculocidal, mycobactericidal
- Virucidal (HBV, HIV, HCV, Herpes, Vaccinia, BVDV, influenza, Ebola, Coronavirus...)
- Does not contain phenols, aldehydes, chlorine or EDTA

Disinfecting properties

ACTIVITY SPECTRUM	STANDARD	STRAINS	CONTACT TIME
BACTERICIDAL * (Dirty conditions)	EN 13727	Pseudomonas aeruginosa Staphylococcus aureus Enterococcus hirae	5 Min.
FUNGICIDAL (Dirty conditions)	EN 13624	Candida Albicans	5 Min.
VIRUCIDAL (Dirty conditions)	DVV ⁽¹⁾ /RKI ⁽²⁾ 2014	BVDV, Vaccinia, HBV, HIV, HCV, Ebola, Herpes, Influenza H1N1, H5N1, Coronavirus	2 Min.
TUBERCULOCIDAL (Dirty conditions)	EN 14348	Mycobacterium Terrae (Surrogate M. tuberculosis)	15 Min.
MYCOBACTERICIDAL (Dirty conditions)	EN 14348	Mycobacterium Terrae Mycobacterium Avium	15 Min.

* Including all the antibiotic resistant strains as MRSA, Klebsiella pneumoniae, Escherichia coli, streptococcus pneumoniae, etc.

(1) DVV: Deutsche Vereinigung zur Bekämpfung der Viruskrankheiten (German Association for the Control of Virus Diseases)

(2) RKI: Robert Koch Institute - German Federal Health Authority

Certifications

- CE mark according to the medical devices Directive (Directive 93/42/EEC)
- Medical device class IIb

For further information, please visit www.medalkan.com

MEDALKAN - Michalakopoulou 102, 115-28 Athens, Greece - Tel: (+30) 210-7484847 - Fax: (+30) 210-7772009 - www.medalkan.com - contact@medalkan.gr



Packaging

- One litre spray bottle (Ref. 20035)

Physical properties

- Appearance: Foaming solution
- Density: 0.99 g/cm³ at 20 °C
- pH: 9.5-10.5 at 20 °C
- Odour: Neutral
- Storage: 5 °C - 35 °C
- Stability: 3 Years
- Biodegradability: According to OCDE 301D

Instructions for use

Spray MEDAPRINT FOAM uniformly on the dental print to be disinfected to ensure a complete wetting. Let the foam act according to the desired disinfection activity (at least 2 min.). Rinse the print with running water and wipe it with a single use paper. Your dental print is clean and disinfected.


Do not mix with other disinfectants.
Store the product between + 5 ° C and + 35 ° C.



Medalkan
HYGIENE & DISINFECTION

MEDAPRINT FOAM[®]

Foaming disinfectant
for dental impressions

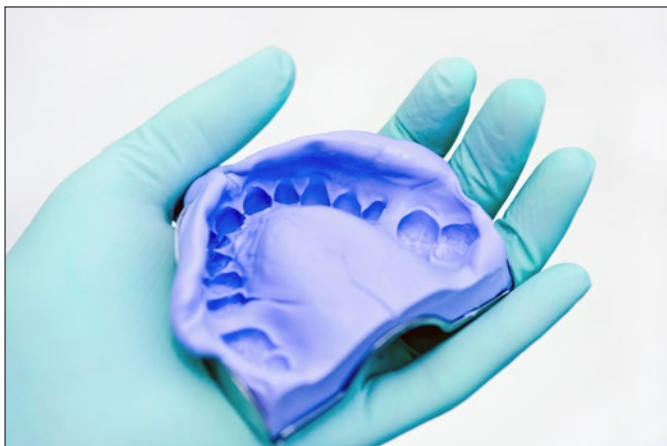

Compatibility

MEDAPRINT FOAM is compatible with impressions from alginate, silicone, polyether and hydrocolloids.

MEDAPRINT FOAM is not compatible with disinfecting preparations containing aldehydes.

Composition

N- (3-aminopropyl) -N dodecylpropano-1,3-diamine,
<5% non-ionic surfactants, corrosion inhibitors, moisturising agent, excipients

MEDAPRINT FOAM is manufactured in the E.U.

MEDALKAN satisfies the requirements of ISO 9001:2015 for quality management system and the requirements of ISO 13485:2016 for the design and manufacture of medical devices.

